

Intended use: AVF is used for temporary cannulation for vascular access for extracorporeal blood treatment.



1.

Inspect packaging and system for any damage or defects. Do not use if external envelope is damaged



2.

Open sterilized packing bag and take product out.



3.

Close clamp.



- 4.**
Remove needle cover and connect the needle tube to the blood vessel



- 5.**
Securely attach the AVF female connector to the Hemodialysis Blood Tubing Set


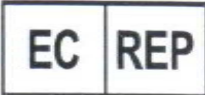















- 6.**
Open clamp. Start priming the set



7. Once finished, close clamp and withdraw needle from vein. Then discard in the sharp collector and adhere to your instruction's policy for safe disposal of all medical waste.

Explanation of the symbols

Symbol	Title of symbol	Description of symbol
	Manufacturer	Indicates the medical device manufacturer
	Authorized representative in the European Community	Indicates the Authorized representative in the European Community
	Date of manufacture	Indicates the date when the medical devices was manufactured
	Use-by-date	Indicates the date after which the medical device is not t be used
	Batch code	Indicates the manufacturer's batch code so that the batch or lot can be identified

	Sterilized using ethylene oxide	Indicates a medical device that has been sterilized using ethylene oxide
	Fragile, handle with care	Indicates a medical device that can be broken or damaged if not handle carefully
	Keep away from sunlight	Indicates a medical device that needs protection from light sources
	Temperature limit	Indicates the temperature limits to which the medical device can be safely exposed
	Do not re-use	Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure
	Non-pyrogenic	Indicates a medical device that is non-pyrogenic
	Do not re-sterilize	Indicates the medical device that is not to be restertilized
	Keep dry	Indicates a medical devices that need to be protected from moisture
	Do not use if package is damaged	Indicates a medical device that should not be used if the package has been damaged or opened
	Consult instruction for use	Indicates the need for the user to consult the instruction for use

PERFECT MEDICAL INDUSTRY (VN) CO., LTD	INSTRUCTION FOR USE	Doc. NO: PF-005	
	A.V.FISTULA NEEDLE SETS	Issue date: 20 May 2023	Version 3
		Page 5 of 5	

Caution:

- Do not use if package is opened or damaged
- Do not use if sterility protectors are loose or missing
- Do not re-sterile product
- Do not allow air to be trapped in set
- Puncturing this set at any point can cause air embolism

Manufactured by:

Perfect Medical Industry (VN) Co., Ltd

Block D7/I, No. 1B Road, Vinh Loc Industrial Zone, Binh Hung Hoa B Ward,
Binh Tan District, Ho Chi Minh City, Vietnam